

AMS SUPERHYDRO

HYDROPHILIC VPS KNEADABLE PUTTY

#PUTSHF002 / BLUE

Made In Germany

PACKAGED SPECIFICALLY FOR



SOFT SuHy / FAST SET BASE & CATALYST IN JARS

Recommendations For Use

Hydrophilic base impression material for the putty/wash technique.

Putty SuHy is used as a hydrophilic base impression material for the putty/wash impression technique in conjunction with **Light SuperHydro**.

Putty SuHy is a kneadable precision impression material based on addition curing vinyl silicone.

Putty SuHy is easy to mix and not sticky. It meets the requirements of ISO 4823 (2000), type 0 (putty consistency).

Application

Perforated or non-perforated metal or plastic trays may be used with **Putty SuHy**. In order to improve adhesion of **Putty SuHy** to the tray, silicone adhesive should be applied as a thin even film. Allow to dry for 3 minutes before loading the impression tray.

Mixing

Use equal parts (volume or weight) of base paste and catalyst paste. **Use the black spoon for the base paste.**

Use the white spoon for the catalyst paste. Knead until the color is homogenous. Load the ready kneaded material into the prepared tray. Insert the loaded tray into the mouth not later than 60 seconds (working time at 23°C or 78°F) after start of mix. Maintain position for at least 2 minutes.

General Notes

Casting - The impression should be stored at normal room temperature (max. 25°C or 78°F). The impression can be poured 30 minutes after removal from the mouth. There are no other time limitations.

Electroplating - All impressions can be copper or silver plated.

Disinfecting - Impressions can be disinfected with standard disinfection solutions.

Storage - Do not store above 25°C (78°F). Do not use after expiration date.

Additional Notes

Do not interchange lids of jars. Do not interchange mixing spoons. Do not use with standard (condensation reaction type) silicone impression materials. Do not use latex gloves as they may interfere with the setting reaction time of the impression material.

Warranty - AmericaSmiles Dental Supply and the manufacturer warrant this product to be free from defects in material and manufacture. AMS Dental Supply makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for the user's application. If this product is defective within the warranty period, your exclusive remedy, and the manufacturer's sole obligation shall be repair or replacement.

Limitation of Liability - Except where prohibited by law, the manufacturer will not be liable for any loss or damage arising from this product, whether direct, indirect, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.



NETWORK

**TO REORDER
CONTACT YOUR DENTAL LABORATORY
OR CALL (708) 367-1207**

**AMERICASMILES DENTAL SUPPLIES
PO BOX 333
CRETE, IL 60417**